

Biosystems A25

CK-NAC

Opt. DGKC / IFCC

Liquid

2 Reagents

REF

Cont.

D94581	5 x 50 ml	4 x 50 ml	Reagent 1
		1 x 50 ml	Reagent 2

D98485	5 x 3 ml	Calibrator	Diacal Auto
D99487	3 x 3 ml	Control abnormal	Diacon CK-MB
D98481	12 x 5 ml	Control normal	Diacon N
D98482	12 x 5 ml	Control abnormal	Diacon P

1. Reagent preparation:

The reagents are ready to use.

2. Instrument settings:

Temperature: 37 °C

GENERAL	Test name	CK-NAC
	Analysis mode	Kinetic
	Sample type	Serum
	Units	U/L
	Reaction type	Ascending
	Decimals	0
	Replicates	1
	Name of assoc. Constituent	-
PROCEDURE	Type of reading	Monochromatic
Volumes	Sample	10
	Reagent 1	200
	Reagent 2	50
	Washing	1.2
	Predilution factor	-
	Postdilution factor	2
	Filters	Main
Reference		
Times	Reading 1	300
	Reading 2	480
	Reagent 2	180
CALIBRATION	Type of calibration	Multiple Calibrator
	Calibrator replicates	3
	Blank replicates	1
	Calibration curve	-
OPTIONS	Blank absorbance limit	0,400
	Kinetic blank limit	-
	Linearity limit	1000

NOTE:

These suggested instructions and instrument parameters are to be used in conjunction with the reagent package insert and the instrument operation manual. Refer to these documents for complete instructions before performing the tests.