

Α ΕΙΣΑΓΩΓΗ ΠΡΩΤΟΚΟΛΛΟΥ ΕΡΓΑΣΤΗΡΙΟΥ ΔΙΑΓΝΩΣΤΙΚΗΣ

HAPTOGLOBIN

Diagnostic reagent for the quantitative in vitro determination of Haptoglobin in human serum by turbidimetric assay

REF

Cont.

A00532 1 x 10 ml Haptoglobin Antibody Reagent
5 x 25 ml PEG6 Buffer

Additionally offered:

A00704 5 x 1 ml Protein Calibrator 5 level series
A00580 1 x 1 ml Protein Calibrator High
A00703 1 x 5 ml Protein Calibrator High
A00590 1 x 1 ml Protein Control
A00800 1 x 5 ml Protein Control
A08591 1 x 1 ml Protein Control Low
A08823 1 x 5 ml Protein Control Low

1. Reagent preparation

Sample: Ready to use

Reagent : Buffer, ready to use

Start Reagent: Antibody Reagent, ready to use

Calibrator: Ready to use

Controls: Ready to use

2. Instrument setting

Test	Haptoglobin
Unit	mg/dL
Decimals	1
Analysis type	Two point end assay 1
Main wavelength	340
Sub wave length	
Method	Turbidimetry
Correction	1.00x + 0
Calibration type	Spline
Standard	Multipoint calibration
Normal range	32 – 205
Sample	10 µL
Automatic dilution	1:10
R1	225 µL
R2	20 µL
Read Main	54
Sub	31
Prozone check	NO

Note: These suggested instructions and instrument parameters are to be used in conjunction with the reagent package insert and the instrument operation manual. Refer to these documents for complete instructions before performing tests.