

APPLICATION PROCEDURE

SYNCHRON CX

GOT (AST, ASAT) MODIFIED IFCC

Liquid
2 Reagents

REF

Cont.

D94610	5 x 100 ml	4 x 100 ml 1 x 100 ml	Reagent 1 Reagent 2	2000 tests 250 µl/test
DB0928	2 x 150 ml	2 x 120 ml 2 x 30 ml	Reagent 1 Reagent 2	1200 tests 250 µl/test
D98485	5 x 3 ml	Calibrator	Diacal Auto	
D98481	12 x 5 ml	Control normal	Diacon N	
D98482	12 x 5 ml	Control abnormal	Diacon P	

1. Reagent preparation:

The reagents are ready to use.

2. Instrument settings:


Test Name:	GOT	User def. No:	#
Chem. Name:	GOT-IFCC		
Reaction Type:	Rate 1		
Unit:	U/l	No of Calibrators	1
Decimal Prec.:	x	Calibrator 1:	*
Reaction Dir.:	negative	2:	
Calculation Factor:	*	3:	
Math. Model:	Linear	4:	
Cal. Time Limit:	336 h	5:	
		6:	
Prim. Wavelength:	340	Sec. Wavelength:	380
Reagent 1 [A] Vol:	200 µl	[B] Vol.:	50 µl
Reagent 2 [C] Vol:	0 µl	Add Time:	32 sec
Sample Volume	20 µl		
Reagent Blank		Reaction	
Start Read:	250 sec	Start Read:	60 secs
End Read:	300 sec	End Read:	240 secs
Usable Range Lower Limit:	0		
Upper Limit:	600		
Error Detection Limits			
Reagent Blk Low:	-1.500	Reaction Low:	- 1.500
High:	1.500	High:	1.500
Substrate Depletion:			
Initial Rate:	-99.999	Delete Abs.:	1.500
Multipoint Span:	-		

#) Data entry by the user

*) Enter calibrator value or factor (the factor has to be checked by calibration or controls)

Bottle Configuration:

A: Reagent 1
B: Reagent 2
C: empty

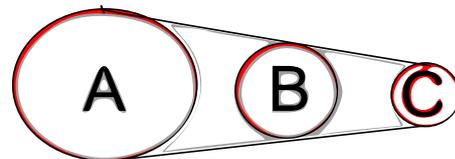

NOTE:

These suggested instructions and instrument parameters are to be used in conjunction with the reagent package insert and the instrument operation manual. Refer to these documents for complete instructions before performing the tests.